



Annex B

Permissions list





The DVC® system can offer different permissions to the different profiles.
These are divided between different sections:

AMS section

AMS View	User can see the Animal Management System the DVC is integrated with
AMS Edit	User can enable/disable the use the Animal Management System integration

Cages

Cage Last Operation View	User can see the Last Operation performed on the specific cage (in the search results)
Cage Last Bedding Change View	User can see the Last Bedding Change performed on the specific cage (in the search results)
Cage Protocol View	User can see in the cage label the assigned Research Protocol (if any)
Cage Protocol Edit	User can edit in the cage label the assigned Research Protocol (if any)
Cage Owner View	User can see in the cage label the assigned Owner (if any)
Cage Owner Edit	User can edit in the cage label the assigned Owner (if any)
Cage Terminate Execute	User can have access to the Termination button to terminate cages through the interface
Cage Add Animal	User can add animals to any cage
Cage Add Generic Animal	User can add Generic animals to any cage
Cage Animal Gender View	User can see the Gender field in the Cage Label
Cage Animal Gender Edit	User can edit the Gender field in the Cage Label
Cage Animal Strain View	User can see the Strain field in the Cage Label
Cage Animal Strain Edit	User can edit the Strain r field in the Cage Label
Cage Animal Date Of Birth View	User can see the Date Of Birth field in the Cage Label
Cage Animal Date Of Birth Edit	User can edit the Date Of Birth field in the Cage Label
Cage Animal Status View	User can see the Health status of the animals
Cage Animal Status Healthy Edit	User can edit the Health status of the animals
Cage Animal Status Sick Edit	User can select the Sickness Reasons from the list created by the Admin
Cage Animal Cull	User can cull animals
Registered Cage Edit	User can edit already registered cages
Cage Animal Sickness Report View	User can see the history of the cage related to the Sickness
Cage Animal Dead Report View	User can see the history of the cage related to the Death
Cage Animal Treatment Edit	User can select the Treatments from the list created by the Admin
Cage Animal Status Note Edit	User can add (if any) notes to the Sickness/Death report



Led

Planned Cage Change Led Execute	User can use the LED in the Bedding Change Task
Planned Daily Check Led Execute	User can use the LED in the Daily Check Task
Alarms Led Execute	User can use the LED to illuminate any cage in alarm
Planned Task Rack Led Execute	User can use the LED to illuminate the first 2 rows of the DVC Rack in the planned task
Termination Navigation Rack Led Execute	User can use the LED during the Termination manual task
Search Cage Led Execute	User can use the LED to illuminate the cages found by the search functionality
Search Rack Led Execute	User can use the LED to illuminate the DVC Rack related to the search functionality

Navigation

Manual Cage Edit	User can use the "Cage Edit" button in the manual task
Manual Cage Split Standard	User can use the "Cage Split Standard" button in the manual task
Manual Cage Split Advanced	User can use the "Cage Split Advanced" button in the manual task
Manual Bedding Change Total	User can use the "Bedding Change Total" button in the manual task
Manual Bedding Change Partial	User can use the "Bedding Change Partial" button in the manual task
Enter Manual Termination	User can use the "Manual Termination" button in the manual task
Cage Search	User can use the "Cage Search" functionality
Enter Manual Reconciliation	User can use the "Reconciliation" functionality

Readers

Reader Manual Association Execute	User can manually associate readers through the Operator Interface
-----------------------------------	--

REM

Rem Configuration	User can configure REM
-------------------	------------------------



 **TECNIPLAST**
innovation through passion